

✓ **REFERENCE**

High Multiparametric calibrator	MPREH-001	1 ml	2-8°C
	MPREH-005	5 ml	2-8°C
Human multiparametric biological fluid standardized from the reference ERM-DA470k/IFCC, sodium azide (< 1g/l)			
Lot #		16J20	
Expiry date		10/2018	
Control date		23/03/2017	
Quality control report #		DGM-QAC-REP-16170	
Document prepared and signed by		L.Ginneberge	

✓ **SAMPLES AND REFERENCE VALUES**

See the corresponding reagents technical sheet.

✓ **COMPOSITION**

The multiparametric high level calibrator is a human biological fluid containing sodium azide (<1g/l) as preservative and the following human proteins: albumin, alpha-1 antitrypsin, alpha-1 glycoprotein acid, alpha-2 macroglobulin, antithrombin III, complement C3, complement C4, ceruloplasmin, haptoglobin, IgA, IgG, IgM, prealbumin and transferrin.

✓ **PRINCIPLE OF TEST**

The human proteins of calibrator react upon a specific antibody for corresponding protein and the turbidity induced by the formation of immune complexes is recorded at appropriate wavelength. The turbidity measured is directly proportional to the antigen concentration of the calibrator which can be used for the quantitative determination of this antigen in immunoturbidimetry.

✓ **PRECAUTIONS**

For in vitro single diagnostic use. To be handled by entitled Personnel. Products from human source were tested and found free from HBsAg and antibodies to HCV and HIV but this material should be treated just as carefully as potentially infective. Products containing sodium azide have to be handled with care; avoid ingestion and contact with skin and mucous membranes. Sodium azide may react with lead or copper plumbing to form highly explosive metal azides.

✓ **ANALYTICAL PERFORMANCES**

See the corresponding reagents technical sheet.

✓ **PREPARATION AND REAGENTS STABILITY**

The calibrator is ready for use; once opened, it is stable until expiry date if stored stoppered in appropriate temperature conditions and without any contamination (avoid pipetting and decantation).

✓ **METHOD OF ANALYSIS AND CALCULATION**

See the corresponding reagents technical sheet.

✓ **QUALITY CONTROL**

Accuracy and reproducibility: analytical performances can be checked with the internal quality control serum of the laboratory or with the Liquichek™ (BIO-RAD) Control sera (see the values range obtained with DiAgam reagents and indicated on the accompanying BIO-RAD sheet).

Calibration: calibration curve and stability of calibration curve can be validated with the DiAgam calibration control (MCCON-002).

In case of analytical performances modification, calibrate the method again and contact the manufacturer if modifications are subsisting.

✓ **BIBLIOGRAPHY**

- (1) Certification of proteins in the human serum. Certified Referenced Material ERM®-DA470k/IFCC. I. Zegers et al. <http://imm.jrc.ec.europa.eu/>
- (2) S. Blirup-Jensen et al. protein standardization V: value transfer. A practical protocol for the assignment of serum protein values from a reference material to a target material. Clin Chem Lab Med (2008); 46(10): 1470-1479.
- (3) G. Merlini et al. Standardizing plasma protein measurements worldwide: a challenging enterprise. Clin Chem Lab Med (2010); 48(11): 1567-1575.



MPREHFTEN 23/03/2017 v04

Proteins:	CAL	H
	g/l	
	certified value	uncertainty
Albumin	96.50	4.83
Alpha1-Antitrypsin**	3.15	0.16
Alpha-1 glycoprotein acid	1.62	0.08
Antithrombin III*	0.83	0.04
Complement C3	3.47	0.17
Complement C4	0.62	0.03
Ceruloplasmin*	0.93***	0.05
Haptoglobin	2.71	0.14
IgA	4.59	0.23
IgG	24.22	1.21
IgM	2.22	0.11
Prealbumin	0.55	0.03
Transferrin	6.43	0.32

The certified uncertainty is the half-width of the 95 % confidence interval of the mean.

Values assigned from the reference ERM-DA470k/IFCC.

**AT-III and Ceruloplasmin are referenced to external controls.*

**** Values compatible with the new Alpha1-Antitrypsin reagent and incompatible with the old reagent. In case of doubt, contact the manufacturer.**

***** Values compatible with the new Ceruloplasmin reagent and incompatible with the old reagent. In case of doubt, contact the manufacturer.**